Testkit ID:

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! IMPORTANT!

Correct registration of your animal including test kit ID on the myFERAGEN platform is a prerequisite for a smooth test procedure. Please follow the steps in these instructions even if you already have a written test request.

Instructions for KIT activation & Sampling

Step 1

myFERAGEN Create account

For quick and smooth test processing, first register on our myFERAGEN platform at my.feragenlab.com. You will also find all test results there later.

Create account under:
<https://my.feragenlab.com/login/register>

Have you already received an account or access data from your breeding club? Then simply login at my.feragenlab.com and continue with the next step.

Step 2

Add dog profile

After successfully logging in, click on the "Add your first dog" button. This will create the profile for the dog on which the genetic analysis is to be carried out. Then save the information you have entered and continue with step 3.

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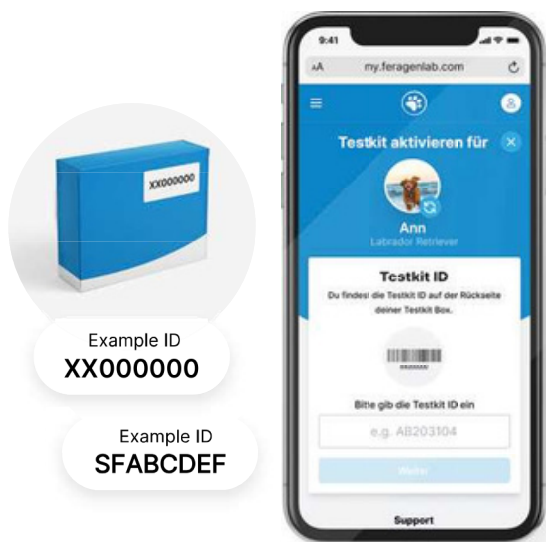
Step 3

Activate test kit

Important: After saving the dog profile, the test kit ID must be assigned to the corresponding profile and activated. This is the only way the laboratory can clearly assign the submitted sample to your dog.

Simply follow the instructions on the platform to activate the test kit ID. You will find the ID either on the packaging of the collection brush or on your invoice.

Step 4

Take the sample

Please request the testkit ID for blood samples in advance by email at support@feragen.at. Your veterinarian will have all the necessary materials on site.

Important note:

You can verify the identity of your animal by having the sample taken by a vet, breed warden or other official person.

Such verified sampling may be necessary for the results to be recognized by breeding associations.

You can find the required form here:

<https://feragen.at/confirmation-sampler>

Step 5

Return shipment of the sample

Pack the brush in the enclosed envelope and send it back to us.

Shipping from Austria
FERAGEN GmbH
Strubergasse 26
5020 Salzburg

Shipping from Germany
KUBEOS GmbH
c/o FERAGEN
Niedervillern 8
83410 Laufen

International shipments
FERAGEN GmbH
Strubergasse 26
5020 Salzburg
AUSTRIA

ATTENTION

The German address is a PO box. The samples are forwarded to our laboratory in Austria three times a week. Please do NOT send your sample by "registered mail", "personal delivery", "registered mail" or "Express" - these shipping methods lead to delays or refusal of acceptance. If you would like to use one of these shipping options, please send the sample directly to our address in Austria.

For shipping outside the EU, you can find all the necessary documents and customs forms to download at:

feragen.at/customs